

WILLI GELLER Creation

CREATION CORRECTION POWDER FIRING GUIDE

CC CORRECTION POWDER FIRING

Preheating Temp.	Drying Time	Raise of Temp	V	Final Temp	Holding Time	Appearance
450°C	6 minutes	45°C/min.	+	770°C	1 min	Shining

ZI-F CORRECTION POWDER FIRING

Preheating Temp.	Drying Time	Raise of Temp	V	Final Temp	Holding Time	Appearance
450°C	4 minutes	45°C/min.	+	690°C	1 min	Shining