

Pasta Rossa

INSTRUCTIONS FOR USE

DESCRIPTION

Pasta Rossa is an indicating paste for the visual control of contact when adjusting dental restorations on the working model. It is petroleum-based and contains color pigments.



USES

– Die Seating

Apply a very thin layer of Pasta Rossa to the die using the applicator brush. Seat the crown on the die, then remove and examine the inside for red marks. Remove any marked areas, and re-check. Continue until no red marks appear.

– Pontic Ridges

Apply a very thin layer of Pasta Rossa to the ridge using the applicator brush. Seat the bridge on the dies, then remove and examine under the pontic(s) for red marks. Remove any marked areas, and re-check. Continue until no red marks appear.

– Partial Clasps

Apply a very thin layer of Pasta Rossa to the partial clasps and/or guide planes using the applicator brush. Seat the partial on the crown (on model), then remove and examine the contours of the crown for red marks. Remove any marked areas, and re-check. Continue until no red marks appear.

CLEANING

Only soap & water or steam cleaner required to clean. Pasta Rossa leaves no residue.

FIRST AID

In case of eye contact, flush eye for several minutes with running water, then contact a physician. If swallowed, contact a physician. For additional information, please refer to MSDS.

STORAGE

Close the container tightly and store in a cool, dry place.